



**DENTISAFE**  
PREMIUM WATER TESTING

## Office Protocol

### Needs to Know

1. Does your office use bottled or municipal water?
2. **Understand the IFU** (Instructions for Use) for your solution.
3. How many waterlines does your opertory have?

### Daily

- **Flush the lines for 20-30 seconds** at the beginning, end, and between patients.
- Follow all protocols required by your solution.

### Monthly

- Use a fast, uncertified, in-office testing method to assess the health of your waterlines with a simple pass/fail result.

### Quarterly

- Send out an **R2a test** to an EPA certified Lab to get an accurate picture of the health of your waterlines.
- Shock **after** sending off samples.

### In Case of Failure

1. Use an in-office indicator to ensure shock worked.
2. Refer to DentiSafe's "**In Case of Failure**" at [DentiSafe.com/Resources](https://DentiSafe.com/Resources).



**EPA Certified Lab #37917**

**CUSTOMER SERVICE**  
**336-688-1400**